



Jobskin HANDY-CARD

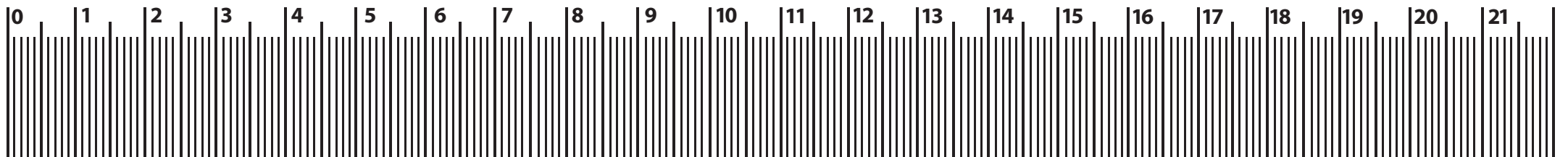

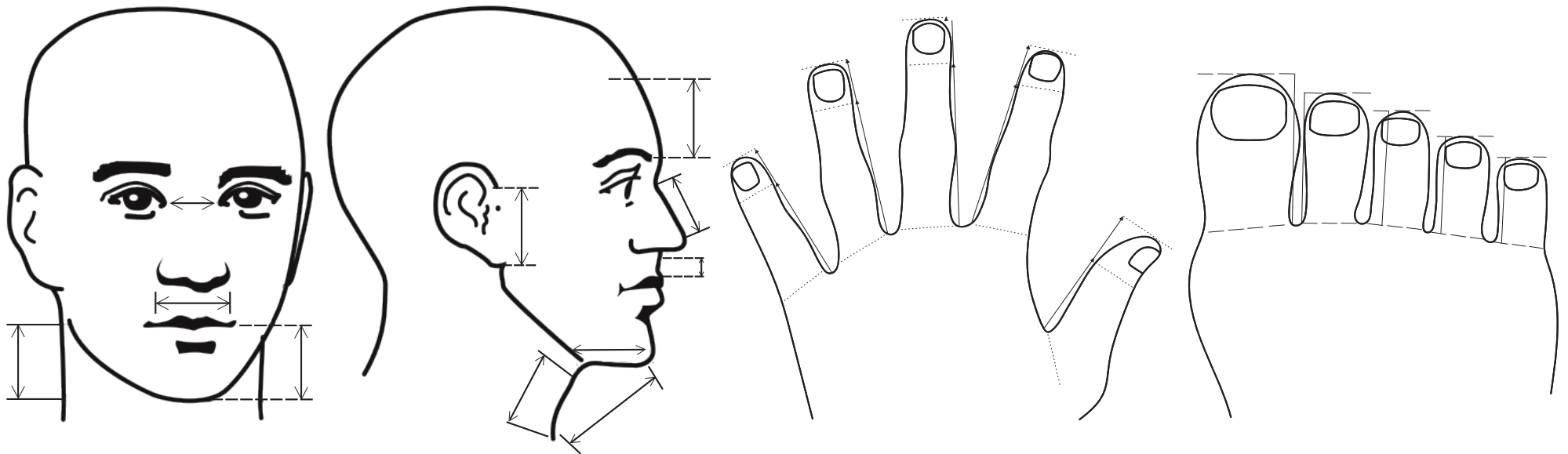

Handy-Tips for taking accurate measurements for fingers and toes

1. Use a pen refill to take the hand or foot trace
2. Mark the base of each webspace before making the outline
3. After taking trace, and before removing the hand or foot from the outline form, mark the desired length of fingers or toes.
4. Use the Jobskin hand/finger tape to take circumferential finger and toe measurements.

Bonus Handy-Tips: The Handy-Card can also be used for marking the height of the 'top arm tape' when measuring for torso garments with armsleeves. Simply tuck the Handy-Card up into the axilla, wrap around the arm, and mark the height of the card.

'The Handy-Card' makes it easier to get some of the face measures as it is more rigid than a tape measure. Refer over for these measurements.

JOBKIN Pty Ltd
Australia
T: +61 3 9915 8000
E: customer.service@jobskin.com
W: www.jobskin.com



TM Trademark of Double Life
All Trademarks acknowledged
JA-001-01

